




THE REAL-CHAIRS ARE CE-CERTIFIED  
IN ACCORDANCE WITH THE MPA STATUTES  
OF MEDICAL DEVICES, LVFS 2003:11.

**EMP**  
LIVING LLC  
www.empliving.com  
info@empliving.com

# FREEDOM OF CHOICE

*options and accessories for REAL 9000 PLUS, REAL 6100  
and REAL 6100 PLUS, Child & Adult*

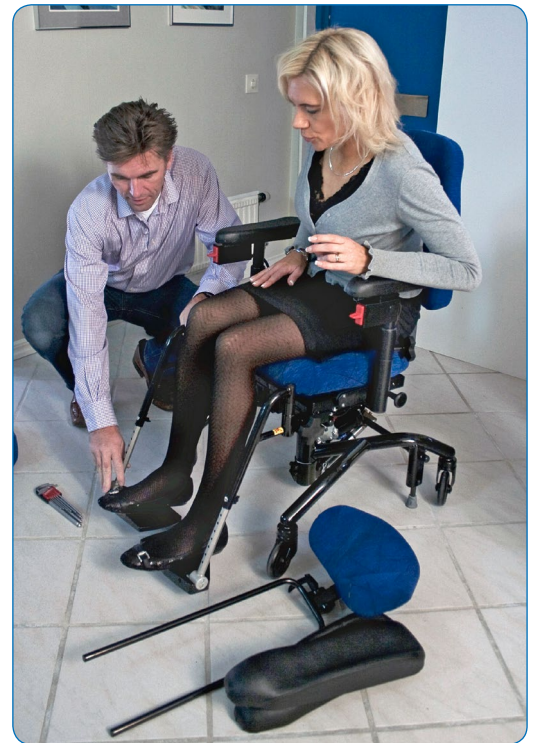

# ACCESSORIES

## BASE

The base is available in several variants in order to satisfy different needs. Our clinic chair base even comes with a built-in attendant foot brake making it easier to stop without having to reach forward for the brake lever. The versatile PLUS-BASE gives extra space for your feet and extensively simplifies the possibility of movement while seated in the chair. An H-BASE gives extra space for your feet and leg supports. The leg supports is preferably to be mounted on H-BASE and may be complemented with calf and knee supports.

## BRAKE LEVER

Brake levers are available in various lengths, and may be placed on either the right or left side. A rheumatic cross can be mounted on the brake lever to simplify braking for people who normally find it difficult to reach the brake lever. Electric brakes are also available.

## BASE EXTENDER

A base extender provides the chair with increased stability and prevents it from tipping. This is a useful option if the user suffers from spasticity.


## CASTORS

The XL-WHEEL 100 mm is standard, and is best suited for linoleum and medium-hard floors. The SL-WHEEL rolls very easily, and has double bearings and works best on hard floors such as parquet and concrete. The BSL-WHEEL is a SL-WHEEL. It rolls very easily, and has double bearings and works best on hard floors such as parquet and concrete. 125- and 150 mm-wheels are well suited for manoeuvring both thresholds and boardwalks.

## FOOT RING

The foot ring serves as support for the feet and makes it easier to maintain an upright posture. The support is easy to fold up or down to match individual requirements, for example when feet need space for transporting. The up-foldable foot ring is available as two models and suitable for all REAL 9000-chairs with PLUS-BASE.

| NO. | NAME   | ART.NO | NO.  | NAME                                       | ART.NO   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|-----|--|--------|--|--|--|----------------------------------|--|---|---------------------|--------|------------------------------|--------|----|------------------------|--------|------------------------------|--------|----|--|--------|---|--|--------|----|---|--|---|---|--|------------------------------|--------|-----------------------------------|--------|------------------------------|--------|---|---|--------|----|-----------------------|--|-----------------------------------|--|--------------------------|-------|---|---|--------|----|----------------|--|--|--|------------------|--------|---|---|--------|-------------------------------------|--------|--|--|------------------------------------|--------|---|----------------------------------|--|---------------|--------|-------|--------|---------------|--------|-------|--------|----|----------------------|--|-------|--------|--|--------|-------|--------|---|--------|---|---|--------|--|--------|-------------------------------------|--------|---|--------|---|---|--------|----|----------------------|--|-------------------------------------|--------|--------------------------------------|--------|--|--|
| 1   | <b>Base 9000 PLUS 48 (adult)</b>                           |        | 8  | <b>Rheumatic cross</b>                     | RC1010   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 100 XL   | 802659 |  | 8  | <b>Brake lever 35 cm with loop f Plus-base</b>                     | BD1110                           |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 100 SL   | 802660 |  |  | 2  | <b>Base 9000 PLUS 43 (child)</b> |  | 9 | <b>Pushbar PLUS</b> | PB1011 | 100 XL* add with brake pedal | 802657 | 10 | <b>Pushhandle PLUS</b> | PB1010 | 100 SL* add with brake pedal | 802658 | 11 | <b>Backrestmech fixed f high back incl. depth adj. Push handle</b> | 803903 | 3 | <b>Base 9000 PLUS Transport incl pushbar and handbrake</b> | 803793 | 12 | <b>Brake lever attachment for PLUS-base</b> |  | 4 | <b>Electric brake for assembly on PLUS-base</b> |  | Adjustable in width, 7 holes | 802800 | Kit for electric chairs, complete | 803276 | Adjustable in width, 2 holes | 802799 | 5 | <b>Four-point brake kit for PLUS-base 43/48</b> | 803037 | 13 | <b>Base extenders</b> |  | including base extenders in front |  | Back for PLUS-base, Pair | BEX20 | 6 | <b>Base 9000 PLUS 48 4-braked 100 XL reversed brake</b> | 804270 | 14 | <b>Castors</b> |  |  |  | Castor XL 100 mm | 801213 | 7 | <b>Electric kit for electric brake, PLUS-base</b> | 802732 | Extra easy rolling castor SL 100 mm | 801144 |  |  | Extra wide easy rolling BXL 100 mm | 803162 | 8 | <b>Brake lever for PLUS-base</b> |  | Castor 125 mm | 800904 | 25 cm | BD1020 | Castor 150 mm | 800905 | 35 cm | BD1010 | 15 | <b>Footrings 1/8</b> |  | 45 cm | BD1030 | Footring 1/8, foldable Right, PLUS-base 48 | FR1010 | 55 cm | BD1040 | Footring 1/8, foldable Left, PLUS-base 48 | FR2010 | 8 | <b>Large ball grip (50mm) for brake lever</b> | LB1010 | Footring 1/8, foldable Right, PLUS-base 43 | FR1020 | <b>Small ball Rubber Black 40mm</b> | LB1040 | Footring 1/8, foldable Left, PLUS-base 43 | FR2020 | 8 | <b>Large ball grip (50mm) for brake lever</b> | LB1010 | 16 | <b>Footrings 1/4</b> |  | <b>Small ball Rubber Black 40mm</b> | LB1040 | Footring 1/4, foldable, PLUS-base 48 | FR3010 |  |  |
| 2   | <b>Base 9000 PLUS 43 (child)</b>                           |        | 9  | <b>Pushbar PLUS</b>                        |  | PB1011                           |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 100 XL* add with brake pedal                               | 802657 | 10   | <b>Pushhandle PLUS</b>                     |  | PB1010                           |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 100 SL* add with brake pedal                               | 802658 |  | 11   | <b>Backrestmech fixed f high back incl. depth adj. Push handle</b> | 803903                           |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 3   | <b>Base 9000 PLUS Transport incl pushbar and handbrake</b> | 803793 | 12   |  | <b>Brake lever attachment for PLUS-base</b>                        |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 4   | <b>Electric brake for assembly on PLUS-base</b>            |        |  | Adjustable in width, 7 holes               | 802800   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | Kit for electric chairs, complete                          | 803276 | Adjustable in width, 2 holes               | 802799                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 5   | <b>Four-point brake kit for PLUS-base 43/48</b>            | 803037 | 13   | <b>Base extenders</b>                      |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | including base extenders in front                          |        |  | Back for PLUS-base, Pair                   | BEX20  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 6   | <b>Base 9000 PLUS 48 4-braked 100 XL reversed brake</b>    | 804270 | 14   | <b>Castors</b>                             |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     |  |        |  | Castor XL 100 mm                           | 801213   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 7   | <b>Electric kit for electric brake, PLUS-base</b>          | 802732 | Extra easy rolling castor SL 100 mm        | 801144                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     |  |        | Extra wide easy rolling BXL 100 mm         | 803162                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 8   | <b>Brake lever for PLUS-base</b>                           |        | Castor 125 mm                              | 800904                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 25 cm  | BD1020 | Castor 150 mm                              | 800905                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 35 cm  | BD1010 | 15   | <b>Footrings 1/8</b>                       |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 45 cm  | BD1030 |  | Footring 1/8, foldable Right, PLUS-base 48 | FR1010   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | 55 cm  | BD1040 |  | Footring 1/8, foldable Left, PLUS-base 48  | FR2010   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 8   | <b>Large ball grip (50mm) for brake lever</b>              | LB1010 | Footring 1/8, foldable Right, PLUS-base 43 | FR1020                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | <b>Small ball Rubber Black 40mm</b>                        | LB1040 | Footring 1/8, foldable Left, PLUS-base 43  | FR2020                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
| 8   | <b>Large ball grip (50mm) for brake lever</b>              | LB1010 | 16   | <b>Footrings 1/4</b>                       |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     | <b>Small ball Rubber Black 40mm</b>                        | LB1040 |  | Footring 1/4, foldable, PLUS-base 48       | FR3010   |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |
|     |  |        | Footring 1/4, foldable, PLUS-base 43       | FR3020                                     |  |                                  |  |   |                     |        |                              |        |    |                        |        |                              |        |    |  |        |   |  |        |    |   |  |   |   |  |                              |        |                                   |        |                              |        |   |   |        |    |                       |  |                                   |  |                          |       |   |   |        |    |                |  |  |  |                  |        |   |   |        |                                     |        |  |  |                                    |        |   |                                  |  |               |        |       |        |               |        |       |        |    |                      |  |       |        |  |        |       |        |   |        |   |   |        |  |        |                                     |        |   |        |   |   |        |    |                      |  |                                     |        |                                      |        |  |  |

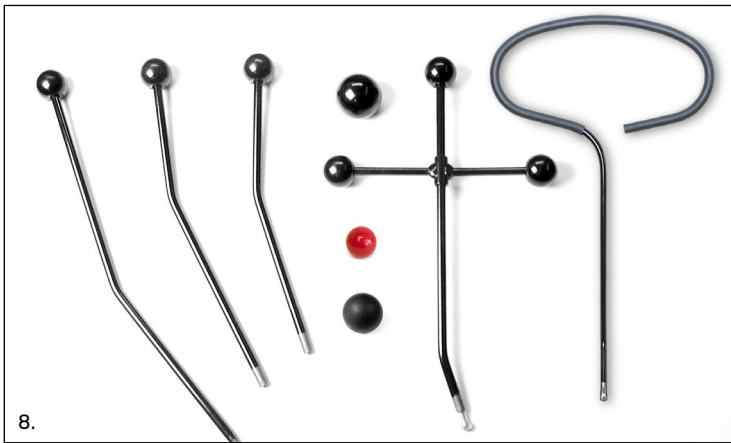

# FOR INDIVIDUAL ADAPTATION

## LEG-SUPPORT MEDIC

Leg-support Medic with whole or divided footplate is available in four sizes: S, M, L and XL. They are height-, angle- and depth-adjustable. Middle-mounted Mobil leg-support is available for adults and children. Leg-supports are primarily mounted on H-bases, and may be complemented with calf- and knee-supports.

## CALF-SUPPORT

The calf-support functions as an extra support for the calves when the leg-supports aren't enough to stabilize the legs. The calf-support is available in different models depending on the leg-support it is easily mounted on.


## KNEE-SUPPORTS

Rectification of legs and feet can be done with adjustable knee-supports. Knee-supports can be regulated into the length of the user's legs and they prevent the user from gliding forward in the chair.

The knee-supports are mounted onto Medic leg-supports and can be pulled out and adjusted in height and depth.

## ADAPTERS

Adapters are available for wheelchair leg-supports: Cross, Adapt, Spinner and Orthopedia, all mounted on the armrests width adjustment arch.

 THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO. | NAME  | ART.NO   | NO.  | NAME   | ART.NO   |
|-----|---|----------|--|--|----------|
| 1   | <b>Leg-support "Comfort" EL</b>                   |          | 8  | <b>Leg-support Medic</b>                     |          |
|     | Right   | 802437   |  | S (18-24 cm)                                 | LS101110 |
|     | Left  | 802436   |  | M (23-32 cm)                                 | LS101210 |
|     | Attachment for leg support "Comfort" EL           | 802979   | L (28-41 cm)                                 | LS101310                                     |          |
|     |   |          | XL (38-51 cm)                                | LS101410                                     |          |
| 2   | <b>Leg-support mobile 6100</b>                    |          | 9  | <b>Leg-support Mini Medic</b>                |          |
|     | S (30-38 cm)                                      | LS202110 |  | S (18-24 cm)                                 | LS111110 |
|     | L (37-53 cm)                                      | LS202210 | M (23-32 cm)                                 | LS111210                                     |          |
| 2   | <b>Leg-support mobile Mini 6100</b>               |          | L (28-41 cm)                                 | LS111310                                     |          |
|     | S (30-38 cm)                                      | LS212110 | XL (38-51 cm)                                | LS111410                                     |          |
|     | L (37-53 cm)                                      | LS212210 |  |  |          |
| 3   | <b>Leg-support 6100 PLUS complete adult angle</b> |          | 10   | <b>Leg-support Mini Medic divided</b>        |          |
|     | electric controlled with rim                      | LS312210 |  | S (18-24 cm)                                 | LS161110 |
| 4   | <b>Leg-support 6100 PLUS complete child angle</b> |          | M (23-32 cm)                                 | LS161210                                     |          |
|     | electric controlled with rim                      | LS312110 | L (28-41 cm)                                 | LS161310                                     |          |
| 5   | <b>Rim for leg-support mobile 6100</b>            | 803211   | XL (38-51 cm)                                | LS161410                                     |          |
|     |   |          |  |  |          |
| 6   | <b>Knee-support</b>                               |          | 11   | <b>Center mounted leg-support</b>            |          |
|     | Right 16x8, black skai                            | KS111038 |  | Compl. 30-38 cm Footplate depth 100 mm       | LS222110 |
|     | Left 16x8, black skai                             | KS121038 | Compl. 37-53 cm Footplate depth 100 mm       | LS222210                                     |          |
| 7   | <b>Knee-support mini</b>                          |          | 12   | <b>Calf-support mechanism, complete</b>      |          |
|     | Right 12x8, black skai                            | KS212038 |  | Pair, Medic legrest, Width 37, cushion 15x13 | CS101038 |
|     | Left 12x8, black skai                             | KS222038 | Pair, Medic legrest, Width 37, cushion 10x10 | CS201138                                     |          |
| 8   | <b>Leg-support Medic divided, Pair</b>            |          | 13   | <b>Adapter for leg-support Cross</b>         |          |
|     | S (18-24 cm)                                      | LS151110 |  | Right  | AD4011   |
|     | M (23-32 cm)                                      | LS151210 | Left   | AD4012                                       |          |
|     | L (28-41 cm)                                      | LS151310 | Pair   | AD4010                                       |          |
|     | XL (38-51 cm)                                     | LS151410 | 14   | <b>Adapter for leg-support Comfort</b>       |          |
|     |   |          |  | Right  | AD2011   |
|     |   |          | Left   | AD2012                                       |          |
|     |   |          | Pair   | AD2010                                       |          |



# SEATS AND SEAT ANGLES

## SEAT TILT

Mercado's gas springs and electric controlled angles can be placed in more locations than just under the seat or armrest. When the seat is tilted, the backrest follows the seat angle and supports the back even when tilted. Contact us for additional alternatives that suits your or your user's needs.

Crank seat control, operated by carers, with an adjustable crank adjustment that can be locked into position. This can be used when gas spring controlled seat angling is not practical. Crank adjustment is suitable for orthopedic clinics where patients should sit at a specific angle, or in school environments for children who need to sit in an active position.

Gas spring controlled seat angle under the seat – the users can easily adjust the seat angle position for themselves for active and passive sitting. When the gas spring control is placed under the armrest it also facilitates standing up from a seated position when the seat is in the forward angled position.

Electric controlled seat angle under armrest is easily accessible for users, and is simple to set in a fixed position. This is a good alternative for users who are unable to adjust their centre of gravity.

## SEATS

The biggest requirement of the seat is to be stable and comfortable. For optimal sitting comfort, the ERGOMEDIC® seats have the right form and softness to ensure optimal stability and comfort, higher at the thigh than the buttocks. Its form distributes pressure evenly and effectively, and prevents the user from sliding forward when the seat is angled.

ERGOMEDIC® PLUS seats have the right form and softness to ensure optimal stability and comfort, higher under the thigh than the buttocks. The higher rounded shape on either side of the thighs means that both the pelvis and the legs are stabilized.

The COXIT seat is divided with individually adjustable flaps. It is used primarily for the alleviation and rehabilitation of orthopaedic injuries to hips or thighs.

ERGOMEDIC SLOW RECOVERY seats are manufactured with two materials in the cushioning, the upper layer comprises of a memory foam that moulds itself and provides good pressure distribution and increased sitting comfort. The lower layer raises the seat bone nodules from the wood plate.



THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO. | NAME  | ART.NO. | NO. | NAME   | ART.NO.          |
|-----|---|---------|-----|--|------------------|
| 1   | <b>Gas spring controlled seat tilt</b><br>Controls under armrest<br>Controls under seat           | SAR31   | 6   | <b>Adjustable strenghtened seat frame 9000, +15°/-23°</b><br>(adjustable in three positions)<br>Adult, width 40-54 cm<br>Child, width 33-39 cm | 803070           |
|     |   | SAR30   |     |  | 803071           |
| 2   | <b>Crank controlled seat tilt under seat</b><br>Complete kit for crank controlled seat tilt       | SAR20   | 6   | <b>Adjustable strenghtened seat frame 6100, +15°/-23°</b><br>(adjustable in three positions)<br>Adult, width 40-54 cm<br>Child, width 33-39 cm | 803072<br>803073 |
| 3   | <b>Prepared for seat tilt</b>   | SAR10   | 7   | <b>Seat frame mech. 9000/EL -15/+15</b><br>Width 29-39 cm, max user weight 75kg  | 803527           |
| 4   | <b>Electric seat tilt</b><br>With maneuverbox<br>With remote<br>With joystick<br>With control box |         | 8   | <b>Wooden seat</b>   | ST10xx90         |
|     |   |         | 9   | <b>Seat with anit-slide fabric</b>   | ST10xx61         |
|     |   |         | 10  | <b>Seat ErgoMedic with hygiene fabric</b>  | STxxxx60         |
|     |   |         | 5   | <b>Electric kit for seat tilt, complete</b>  | 803275           |



Contact Mercado or view the brochure 'Seatsystems' for more information.



# BACKS AND BACKREST MECHANISMS

## BACKREST MECHANISM

MEDIC BACKREST MECHANISM provide the option of increasing the seating depth in a more effective manner than with a standard backrest mechanism, this is necessary if the user is severely kyphotic. In order to keep the user's head up, the back rest is adjusted to a backwards leaning position, and the back plate angled to provide the best possible support so that the user's upper body is prevented from falling forward. The advantage of the Medic backrest mechanism is that the angle adjustment is located on the lower edge of the back plate meaning that angle-adjustment of the back plate does not limit seat depth, and also provides back support. It automatically centers the user from the side because the back support is distributed along the width of the back along the bottom edge.

The COMFORT BACKREST MECHANISM can be adjusted in height, depth and angle by using the gas spring control. The backrest becomes yieldingly soft, and the user can change seat depth themselves whilst sitting in the chair. LOW BACKREST is an ergonomically correct back plate that provides good support for the lower back, and supports the curve in the small of the back. The low backrest allows for extended freedom of movement in the upper body.

HIGH BACKREST provides even more support and stability, especially upwards, but is not so high that upper body and shoulder movement is restricted. The high backrests are also available with extended lower back support. EXTRA HIGH BACKREST provides good lower back support and back plate support higher up in the central back area (between the shoulder blades). The backrest tapers off at the top so that shoulder blades are not impeded in work situations.

## BACKS

The ERGOMEDIC backs shape and angle of the high back has been improved to create more space for the pelvis and its softer parts. The lower edge of the back panel now has a gentle 'U' shape and is softer. The ERGOMEDIC® PLUS back is available in several variants. The higher backrest provides more shape around the pelvis and thoracic spine. At the top in the centre, between the shoulder blades, a soft extra support has been integrated. This to enable the user to sit upright and stable against the backrest, whilst at the same time leaving enough room for free movement of the shoulders. The high backrest is also available with extra side wedges.



The low backrest has a similar shape to the high one, but is designed for supporting the pelvis and lumbar spine.



THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO. | NAME   | ART.NO. |    |   |
|-----|--|---------|----|---|
| 1   | Standard backrest mechanism, Low back          | BRM1000 | 6  | <b>Comfort backrest mechanism, High back</b>        |
|     |  |         |    | Shallow (for kids and coxitseat) 801742             |
|     |  |         |    | Deep 801744   |
| 2   | Standard backrest mechanism, High back         | BRM4000 | 7  | <b>Elreglerad backrest mechanism with Low back</b>  |
| 3   | <b>Medic backrest mechanism, Low back 112</b>  |         |    | Electric kit + Relay kit for backrest angle 803277  |
|     | S 200 mm                                       | BRM1800 |    | Back pole Std. Cla/ErgoM Low back 801265            |
|     | L 240 mm                                       | BRM1900 | 8  | <b>Elreglerad backrest mechanism with High back</b> |
|     | XL 340 mm                                      | BRM2000 |    | Electric kit + Relay kit for backrest angle 803277  |
|     |  |         |    | Back pole Std. Cla/ErgoM High back 800841           |
|     | <b>Medic backrest mechanism, Low back 155</b>  |         | 9  | <b>Electric kit for backrest angle</b> 802731       |
|     | S 200 mm                                       | BRM1500 | 10 | <b>Double Medic backrest mechanism</b>              |
|     | L 240 mm                                       | BRM1600 |    | Attachment for Medic backrest mechanism 802859      |
|     | XL 340 mm                                      | BRM1700 |    | Backrest mechanism, L 155mm (x2) 801082             |
| 4   | <b>Medic backrest mechanism, High back 112</b> |         |    | Backrest mechanism, L 112mm (x2) 801083             |
|     | S 200 mm                                       | BRM4800 |    | Backrest mechanism, XL 155mm (x2) 801084            |
|     | L 240 mm                                       | BRM4900 |    | Backrest mechanism, XL 112mm (x2) 801085            |
|     | XL 340 mm                                      | BRM5000 |    | Back pole Medic double 802860                       |
|     |  |         |    | <b>Backrest mechanism double Medic complete</b>     |
|     | <b>Medic backrest mechanism, High back 155</b> |         |    | L 155 mm, H 240 High back BRM5600                   |
|     | S 200 mm                                       | BRM4500 |    | L 155 mm, H 340 High back BRM5700                   |
|     | L 240 mm                                       | BRM4600 |    |   |
|     | XL 340 mm                                      | BRM4700 |    |   |
| 5   | <b>Comfort backrest mechanism, Low back</b>    |         |    |   |
|     | Shallow (for kids and coxitseat)               | 801743  |    |   |
|     | Deep   | 801745  |    |   |



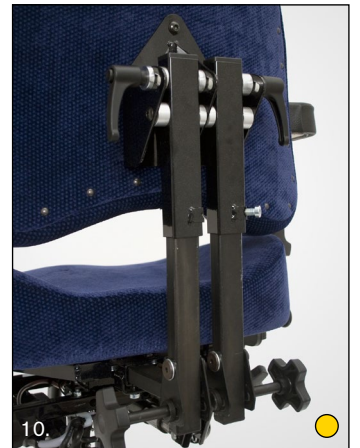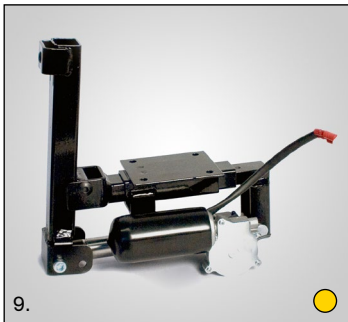

View the brochure 'Strength and Function' for more information about the electric controlled backrest angle.



Contact Mercado or view the brochure 'Seatsystems' for more information.

# ARMRESTS AND ARMRESTS MECHANISMS

## ARMREST

The armrest is an extremely important part of the ERGOMEDIC system, so we have also developed these to be more ergonomic and comfortable. The edges have been beveled, both on the inside and the outside to minimize the pressure against the underarms during prolonged contact. The armrest panels are hollowed out at the back in order to stabilize the elbows. At the front they have been designed to provide a good grip when standing up and transferring pressure.

The comfort armrest is a slightly wider arm support with softer stuffing, which allows the user to easily rest their arm on the armrest.

The hemiplegi armrest is suitable for partially paralyzed, or users with restricted control. The armrest panel has been hollowed out, which makes it easier to keep the paralyzed arm on the rest. The armrest panel also rotates, which allows for the best possible position.

The ERGOMEDIC® PLUS armrests are shaped so the arms are supported from the sides, which means that the upper body is more relaxed. They are also raised beneath the wrists to provide extra relief.

## ARMREST MECHANISM

Armrest mechanism 9000 PLUS is adjustable in height. They can be lowered totally, so that movement sideways is possible for the user.


Armrest hemiplegi 9000 PLUS is suitable for users with one-side paralysis. The mechanism makes the armrest pivotal sideways for best possible positioning and it is possible to lock the hemiplegi mechanism in a certain position.

Armrest mechanism 6100 exist as both angle settable and non-angle settable. The user may adjust the armrest in height and width him-/herself. The shedable armrests can be tilted backwards, which makes movement sideways possible in an easy way.

Armrest mechanism 6100 hemiplegi originate when the armrest mechanism 6100 is completed with the hemiplegi mechanism for shedable armrests. In this case the operation results in an angular adjustment to the side.

● THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO. | NAME  | ART.NO.                     | NO. | NAME   | ART.NO.                              |
|-----|---|-----------------------------|-----|--|--------------------------------------|
| 1   | <b>Armrest ErgoMedic</b><br>25 cm<br>30 cm<br>35 cm<br>40 cm                  | <b>For foldable armrest</b> | 6   | <b>Armrest Hemiplegi</b>   | AR406038                             |
|     |   | <b>For regular armrest</b>  |     |  |                                      |
|     |   | AR151038 AR101038           |     |  |                                      |
|     |   | AR152038 AR102038           |     |  |                                      |
|     |   | AR153038 AR103038           |     |  |                                      |
| 2   | <b>Armrest Mobile</b><br>25 cm<br>30 cm<br>35 cm<br>40 cm                     | <b>For foldable armrest</b> | 7   | <b>Armrest</b><br>Armrest 9000 PLUS Right, 32 cm<br>Armrest 9000 PLUS Right, 38 cm<br>Armrest 9000 PLUS Left, 32 cm<br>Armrest 9000 PLUS Left, 38 cm | 802700<br>802701<br>802702<br>802703 |
|     |   | <b>For regular armrest</b>  |     |  |                                      |
|     |   | AR301038 AR311038           |     |  |                                      |
|     |   | AR302038 AR312038           |     |  |                                      |
|     |   | AR303038 AR313038           |     |  |                                      |
| 3   | <b>Armrest Comfort</b><br>30 cm<br>35 cm<br>40 cm                             | <b>For regular armstöd</b>  | 8   | <b>U-bar for width adjustment PLUS Hö</b>  | 802698                               |
|     |   | AR502038                    |     |  |                                      |
|     |   | AR503038                    |     |  |                                      |
|     |   | AR504038                    |     |  |                                      |
|     |   | AR504038                    |     |  |                                      |
| 4   | <b>Armrest ErgoMedic PLUS</b><br>42 cm Hö<br>42 cm Vä<br>30 cm Hö<br>30 cm Vä | <b>For regular armrest</b>  | 9   | <b>U-bar for width adjustment PLUS Vä</b>  | 802699                               |
|     |   | AR205138                    |     |  |                                      |
|     |   | AR205238                    |     |  |                                      |
|     |   | AR202138                    |     |  |                                      |
|     |   | AR202238                    |     |  |                                      |
| 5   | <b>Armrest Mini</b><br>25 cm<br>30 cm   | <b>För regular armrest</b>  | 10  | <b>Armrest 6100 for chair without seat tilt</b><br>Left<br>Right   | ARM3012<br>ARM3011                   |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
| 6   | <b>Armrest Mini</b><br>25 cm<br>30 cm   | <b>För regular armrest</b>  | 11  | <b>Armrest mechanism 6100 for chair with seat tilt</b><br>Left<br>Right  | ARM3112<br>ARM3111                   |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
| 7   | <b>Armrest Mini</b><br>25 cm<br>30 cm   | <b>För regular armrest</b>  | 12  | <b>Hemiplegic mechanism for foldable armrests</b><br>Pair<br>Right<br>Left   | 802991<br>802989<br>802990           |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
| 8   | <b>Armrest Mini</b><br>25 cm<br>30 cm   | <b>För regular armrest</b>  | 13  | <b>Armrest hemiplegic 9000 PLUS 32 cm</b><br>Right<br>Left   | 802992<br>802993                     |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |
|     |   | AR111038                    |     |  |                                      |
|     |   | AR112038                    |     |  |                                      |

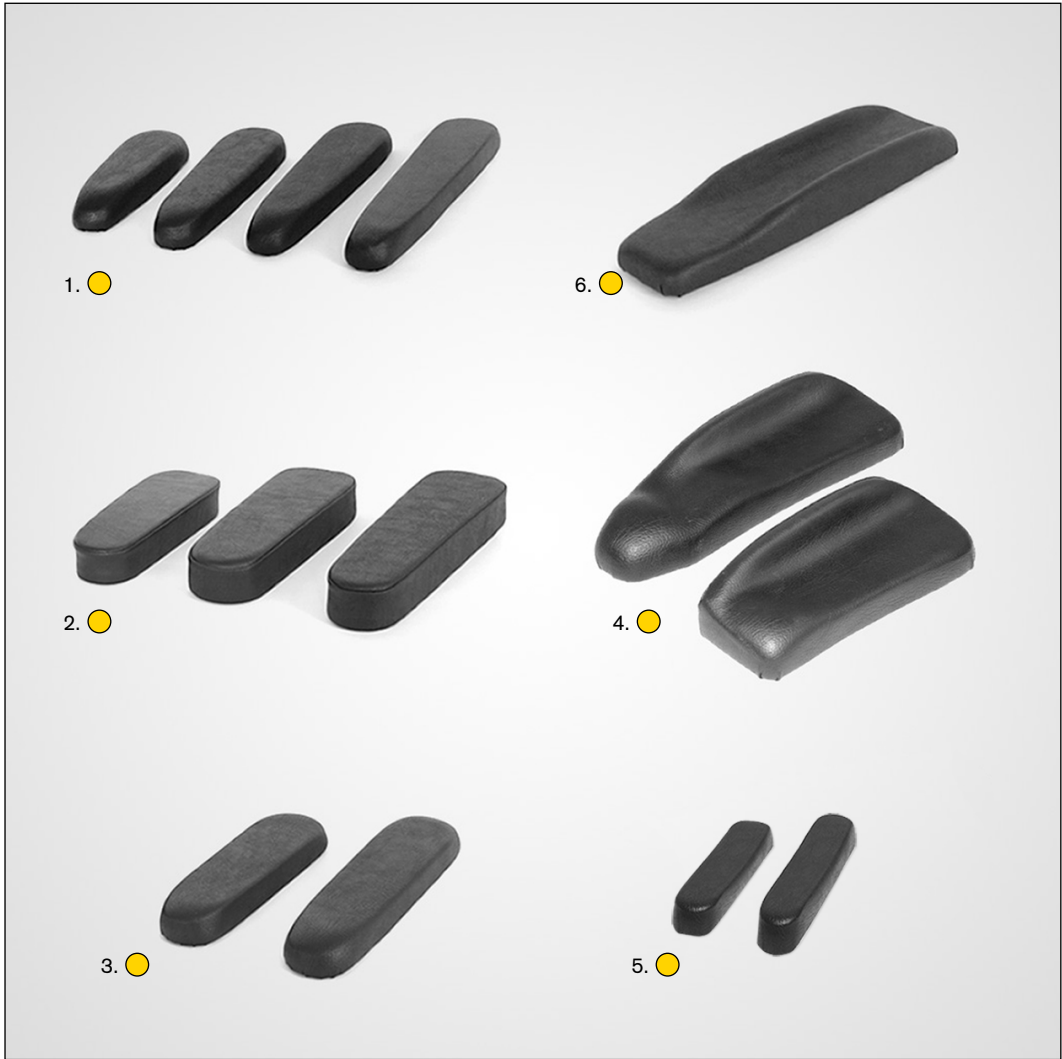

For info on the armrest for the wire controls, contact Mercado or visit [www.mercado.se](http://www.mercado.se)



# MANY ACCESSORIES PROVIDE INCREASED FLEXIBILITY

## TRUNK SUPPORT

Trunk support is used in order to achieve a more upright and active sitting position. Mercado has three types of trunk support: height and depth adjustable trunk support, straight trunk support and curved trunk support.

## TABLE

A smoke coloured transparent lamina with side edges. The table is tip-up and compatible with the REAL 9000 programme. The measurements are 60x45 cm and mounts on the right armrest.

## TRAY

A tray for unloading. It is made out of wood and has the measurements 45x30 cm. The tray turns sideways and can be mounted on optional side of the chairs armrests. This product is also compatible with the REAL 9000 programme.

## HIP BELT

A hip belt eliminates the risk of the user falling forward from the chair. It is available in sizes S 51-71 cm, M 68-83 cm and L 83-103 cm.

## AMPUTATION SUPPORT

The amputation support helps keeping the leg in the right position and is designed to be comfortable for the amputated leg to rest in. The amputation support is available for both legs and it is mounted with a fastener under the seat.

## LEG DIVIDER

A leg divider is used when wanting to prevent the knees from being in contact with each other. The leg divider provides with increased stability and sitting posture. A leg divider is available in two sizes, 11 cm and 15 cm. The mechanism for the leg divider is available in three lengths, child – 57 mm, adult – 115 and 175 mm.




## COVER

The covers protect the upholstery from dirt and dust, as well as extending the chairs lifetime. Covers are available in different colours for a variety of combinations.

Incontinency covers are a quick and easy solution to protect seats against moisture.

● THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO.                                 | NAME   | ART.NO.                      | NO.  | NAME                                      | ART.NO. |
|-------------------------------------|--|------------------------------|--|---|---------|
| 1                                   | <b>Height and depth adjustable trunk support</b> |                              | 8  | <b>Leg divider attachment</b>             |         |
|                                     | Trunk support mechanism adjustable Right         | TS31                         |  | 57 mm complete                            | LD15    |
|                                     | Trunk support mechanism adjustable Left          | TS32                         |  | 115 mm complete                           | LD16    |
|                                     | Cushion (height and depth adjustable)            | TS2038                       |  | 175 mm complete                           | LD17    |
| 2                                   | <b>Straight trunk support</b>                    |                              | 9  | <b>Hipbelt with attachment and screws</b> |         |
|                                     | Trunk support mechanism straight Right           | TS11                         |  | S (51-71 cm)                              | HB1010  |
|                                     | Trunk support mechanism straight Left            | TS12                         |  | M (68-83 cm)                              | HB1020  |
|                                     | Cushion (grey plush)                             | TS1046                       |  | L (83-103 cm)                             | HB1030  |
| 3                                   | <b>Curved trunk support</b>                      |                              | <b>Hipbelt without attachment</b>                                    |   |         |
|                                     | Curved trunk support mechanism Right             | TS21                         | S (51-71 cm)   | 800952                                    |         |
|                                     | Curved trunk support mechanism Left              | TS22                         | M (68-83 cm)   | 800953                                    |         |
|                                     | Cushion (grey plush)                             | TS1046                       | L (83-103 cm)  | 800954                                    |         |
| 4                                   | <b>Table, smoke color 60x45 cm*</b>              | TA1010                       |  | Attachment for hipbelt, Pair              | 800878  |
|                                     | f PLUS armrest                                   |                              | 10   | <b>Amputation support</b>                 |         |
| <b>Table, smoke color 60x45 cm*</b> | TA1030   | Amputation support mechanism |  | 800827                                    |         |
|                                     | f foldable armrest                               |                              | Attachment   | 800786                                    |         |
| 5                                   | <b>Table, smoke color 45x37 cm*</b>              | TA1015                       | Amputation support cushion (grey plush)                              | AS1046                                    |         |
|                                     | f PLUS armrest                                   |                              | 11 <b>Covers are available for all upholstery details</b>            |   |         |
|                                     | <b>Table, smoke color 45x37 cm*</b>              | TA1035                       | 12 <b>Bracket for H-Harness including screws</b> BH2540              |   |         |
|                                     | f foldable armrest                               |                              | Bracket Upper for H-Harness  | 804184                                    |         |
| 6                                   | <b>Tray 45x30 cm with attachment</b>             | TA1020                       | Note: Bracket Upper are sold individually                            |   |         |
|                                     |  |                              | 7 <b>Leg divider cushion (black skai)</b>                            |   |         |
|                                     | 11 cm  | LD1038                       | *To be combined with non-foldable armrestmechanism PLUS on REAL 6100 |   |         |
|                                     | 15 cm  | LD2038                       |  |   |         |

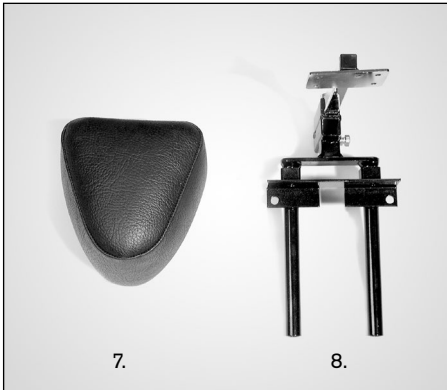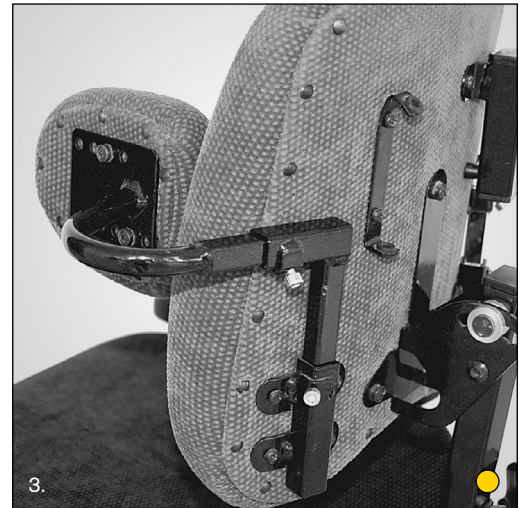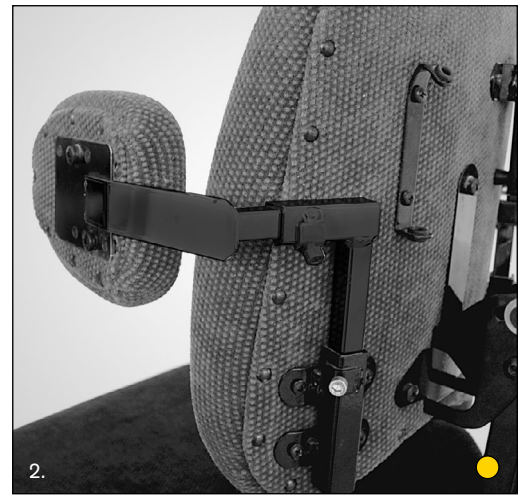

See [www.mercado.se](http://www.mercado.se) or contact Mercado for more information regarding covers for upholstery details, where you also can find information regarding the upholstery and available colors.

# UPHOLSTERY AND ELECTRIC CONTROLS

## NECK SUPPORT

Neck support provides support to the head in order to obtain the best seating position, as well as to provide relief for the neck and back when trunk control is poor. It can easily be adjusted to obtain the optimal setting. Available in flat, curved and ERGOMEDIC PLUS cushions.

## SIDE SUPPORTS

Side supports help the user maintain stability and position. When used for thigh support, feet are better supported on the footplates or floor, and it prevents legs from sliding apart. It can be adjusted for height and width, or turned backwards. Available in four sizes: XS, S, M and L.

## CONTROLS FOR SEAT HEIGHT

Controls for raising and lowering the seat height can be placed under the armrest, on the backrest with an attachable box or be controlled by a joystick. A remote control is also available for use in clinics.

## ELECTRIC BRAKE






Electric brakes facilitate braking for users that have problems using manual brake levers. For the electric brake there is a red button located on the outside of the electric lifting control under the armrest. A single push of a button can take care of the entire braking procedure.


## UPHOLSTERY

The chair is delivered in plush, skai or patterned fabric in a number of colours, all of which are machine washable.

☉ THE ACCESSORY IS ALSO APPLICABLE FOR REAL 6100

| NO. | NAME   | ART.NO. | NO. | NAME   | ART.NO. |
|-----|--|---------|-----|--|---------|
| 1   | Neck support cushion flat, 29x16 cm              | NS1046  | 10  | Joystick with brake button for EL-chairs                   |         |
|     |  |         |     | 2-function   | 802742  |
|     |  |         |     | 3-function   | 802743  |
| 2   | Neck support cushion curved, 31x18 cm            | NS3046  |     |  |         |
| 3   | Neck support cushion EM PLUS 30x23               | NS2046  | 11  | Remote controls  |         |
|     |  |         |     | Multi (2-4 functions)                                      | 801808  |
| 4   | Neck support cushion EM PLUS 30x8                | NS4046  |     | Up/down (1 function)                                       | 801806  |
| 5   | Neck support mechanism universal                 |         | 12  | Free standing electric control (1 function)                | 802739  |
|     | S (h 28 cm)                                      | NS30    |     |  |         |
|     | M (h 38 cm)                                      | NS20    | 13  | Free standing electric control (2-4 function)              | 802740  |
|     | L (h 48 cm)                                      | NS10    |     |  |         |
| 6   | Neck support mechanism PLUS laterally adjustable |         | 14  | Footpedal for electric seat height                         | 802688  |
|     | S (h 28 cm)                                      | NS31    | 15  | Battery charger for electric chair                         | 802935  |
|     | M (h 38 cm)                                      | NS21    | 16  | Battery f REAL electric chair 12v 7,2 Ah                   | 801134  |
|     | L (h 48 cm)                                      | NS11    |     |  |         |
| 7   | Side support attachment PLUS                     |         | 17  | Batteri f SC-EL children chair 3 Ah                        | 801135  |
|     | Right  | SS11    |     |  |         |
|     | Left   | SS12    | 18  | Batterycharger 6100 Plus 4 amp (17V start)                 | BAC1011 |
|     | Pair   | 802874  |     |  |         |
| 8   | Side support cushion                             |         | 19  | Batterypack Mobil 6100 PLUS 24 V<br>Sonnenschein A512/25G5 | BA1010  |
|     | XS (15x8 cm)                                     | SS10XX  |     |  |         |
|     | S (28x8 cm)                                      | SS20XX  | 20  | Controlbox "flat" PLUS for up/down                         | 803893  |
|     | M (25x14 cm)                                     | SS30XX  |     |  |         |
|     | L (30x16 cm)                                     | SS40XX  | 21  | Controlbox PLUS for up/down, with cabling                  | 803034  |
| 9   | Charge box                                       |         |     |  |         |
|     | 1-function                                       | 802212  |     |  |         |
|     | 4-function                                       | 802213  |     |  |         |

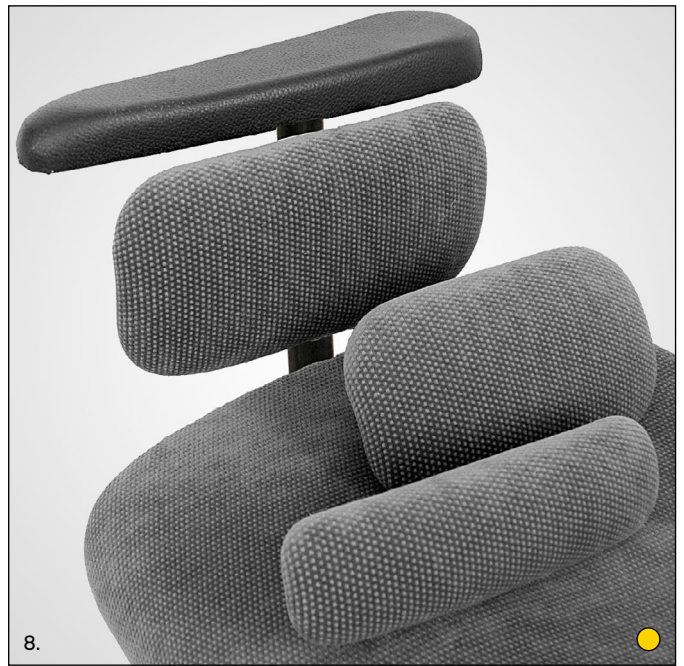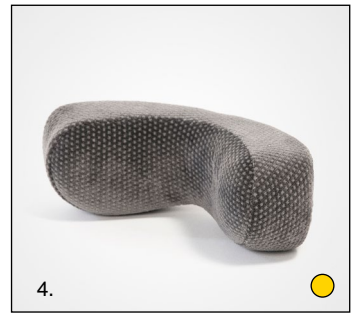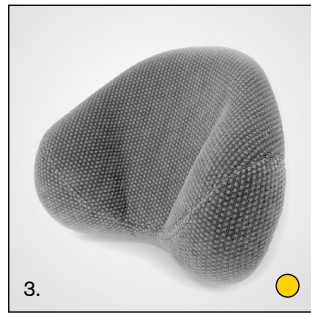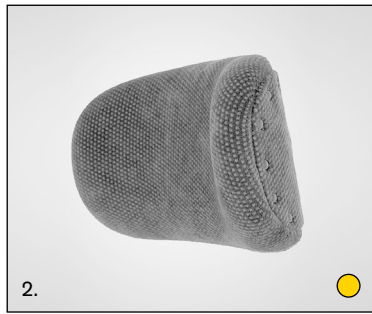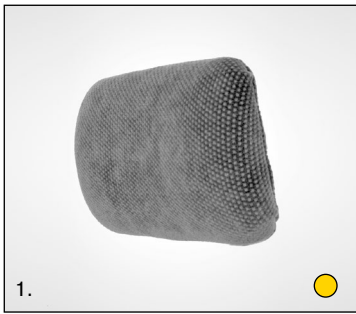

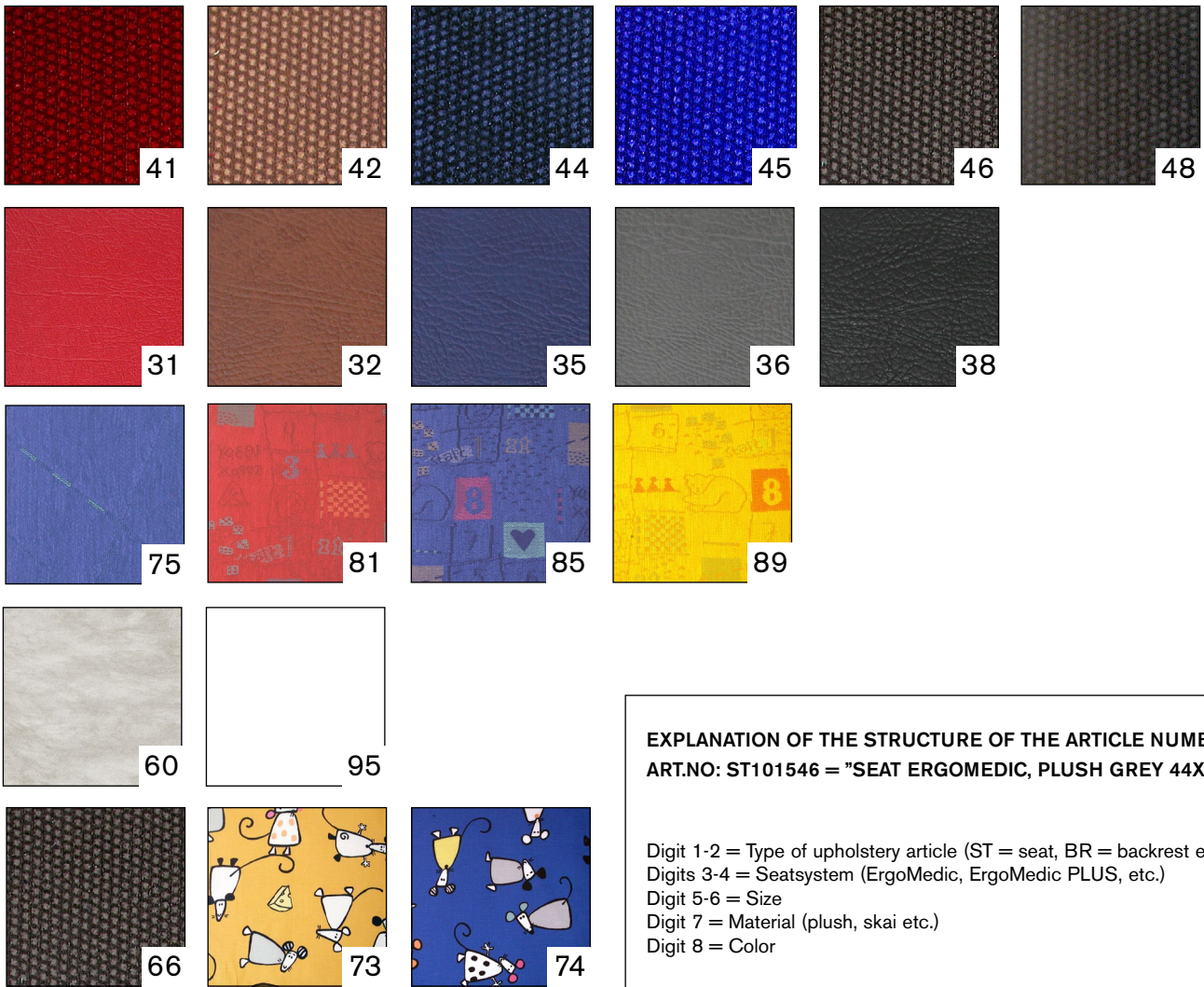

**EXPLANATION OF THE STRUCTURE OF THE ARTICLE NUMBER**  
**ART.NO: ST101546 = "SEAT ERGOMEDIC, PLUSH GREY 44X48 CM"**

Digit 1-2 = Type of upholstery article (ST = seat, BR = backrest etc.)  
 Digits 3-4 = Seatsystem (ErgoMedic, ErgoMedic PLUS, etc.)  
 Digit 5-6 = Size  
 Digit 7 = Material (plush, skai etc.)  
 Digit 8 = Color

**UPHOLSTERY**

**Plush**

41 Bordeaux    42 Brown    44 Marine blue    45 Royal blue    46 Grey    48 Black

**Skai**

31 Red    32 Brown    35 Blue    36 Grey    38 Black

**Patterned fabric, Adult**

75 Blue

**Patterned fabric, Child**

81 Red    85 Blue    89 Yellow

**Alternative covering**

60 Hygiene fabric    95 Without clothing

**Covering**

66 Incontinence cover, grey plastic plush    73 Child/Yellow cotton    74 Child/Blue cotton

Coverings are available for all of Mercado Medic's upholstered parts,  
 visit [www.mercado.se](http://www.mercado.se) or contact us for more information.



**Mercado Medic AB**  
 Tryffelslingan 14  
 181 57 Lidingö  
 Sweden

**Phone & Fax**  
 Ph + 46 8 555 143 00  
 Fax + 46 8 555 143 99

**Email & Internet**  
 Email [info@mercado.se](mailto:info@mercado.se)  
 Internet [www.mercado.se](http://www.mercado.se)