




2014

ERGONOMIC SEAT SYSTEMS

for the REAL-chairs

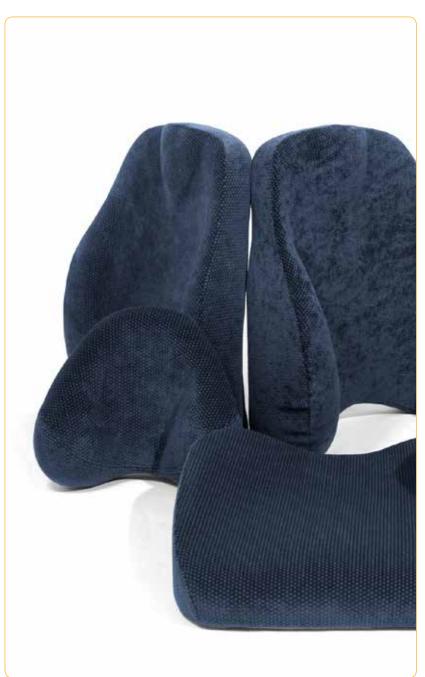









ErgoMedic®

UPHOLSTERY FOR REAL 9000, 9000 PLUS, 6100, 6100 PLUS

With various backs, seats, castors, lifting columns and accessories, it is easy to put a functional and comfortable chair together. To further increase sitting comfort, Mercado Medic has collaborated with registered physiotherapist, Bengt Engström, to develop the ErgoMedic seat system for the REAL 9000. ErgoMedic consists of three interacting parts: Seat, Back and Armrest.

BACKREST

The ErgoMedic back rest and seat work together to provide the most comfortable, and effective sitting position. The shape and angle of the high back has been improved to create more space for the pelvis and its softer parts. The lower edge of the back panel now has a gentle 'U'-shape and is softer. This, combined with the shape of the seat, provides better sitting depth and a more comfortable position for the lower back and pelvis.

SEAT

For optimal sitting comfort, the ErgoMedic-seats have the right form and softness to ensure optimal stability and comfort, higher at the thigh than the buttocks. Its form distributes pressure evenly and effectively and prevents the user from sliding forward when the seat is angled. The back edge of the seat is set low to increase the sitting depth. This means that the seat bone nodules can move backwards when the users stretch out forwards. When the pelvis is positioned at the very back of the seat, and well supported by the backrest, the users legs are also stabilized.

COXIT SEAT


Mercado's Coxit seat has the same high ergonomic quality as the ErgoMedic seat. The Coxit seat is divided with individually adjustable flaps.

It is used primarily for the alleviation and rehabilitation of orthopaedic injuries to hips or thighs.

ARMREST

The armrest is an extremely important part of the ErgoMedic system, so we have also developed these to be more ergonomic and comfortable. The edges have been bevelled, both on the inside and the outside to minimize the pressure against the underarms during prolonged contact. The armrest panels are hollowed out at the back in order to stabilize the elbows. At the front they have been designed to provide a good grip when standing up and transferring pressure.

The armrests are covered with soft imitation leather that is more pleasant to the touch and is less slippery if the armrest becomes damp.

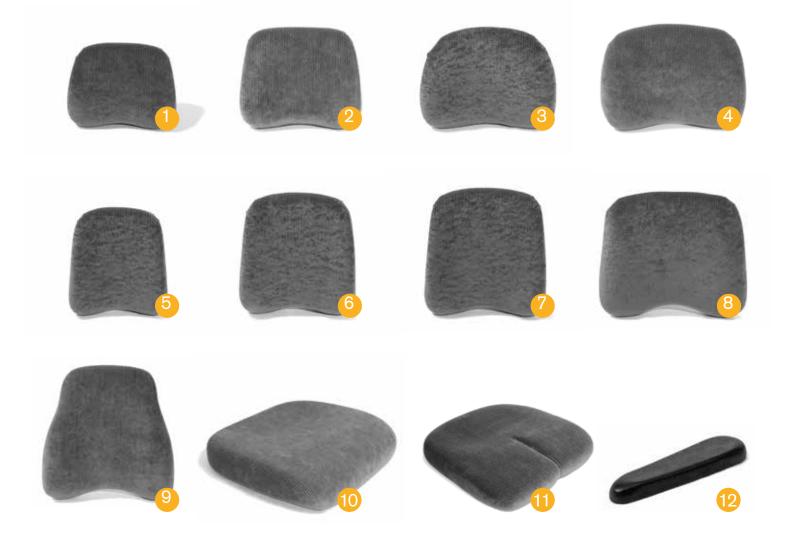

Bac	krest ErgoMedic, width x depth	Article No.	Sea	Coxit ErgoMedic, width x depth	Article No.
1	Backrest low ErgoMedic 27x22 cm	BR107046	11	Seat ErgoMedic Coxit 40x43 cm	ST133046
	Backrest low ErgoMedic 23x27 cm	BR105046	11	Seat ErgoMedic Coxit 40x48 cm	ST132546
	Backrest low ErgoMedic 23x20 cm	BR108546	11	Seat ErgoMedic Coxit 44x43 cm	ST132046
2	Backrest low ErgoMedic 30x26 cm	BR106546	11	Seat ErgoMedic Coxit 44x48 cm	ST131546
3	Backrest low ErgoMedic 33x26 cm	BR106046	11	Seat ErgoMedic Coxit 48x48 cm	ST131046
4	Backrest low ErgoMedic 38x26 cm	BR105546		<u> </u>	
	-		Arm	rest ErgoMedic	
5	Backrest high ErgoMedic 27x32 cm	BR104546	12	Armrest ErgoMedic 25 cm	AR101038
6	Backrest high ErgoMedic 30x34 cm	BR103546	12	Armrest ErgoMedic 30 cm	AR102038
7	Backrest high ErgoMedic 33x36 cm	BR102546	12	Armrest ErgoMedic 35 cm	AR103038
8	Backrest high ErgoMedic 40x34 cm	BR103046	12	Armrest ErgoMedic 40 cm	AR104038
9	Backrest high ErgoMedic 42x43 cm	BR101546		-	
	Backrest high ErgoMedic 46x43 cm	BR101046			
Sea	t ErgoMedic, width x depth	Article No.		- THE STRIC	TURE OF
10	Seat ErgoMedic 29x32 cm	ST105546		EXPLANATION OF THE STRUC	
10	Seat ErgoMedic 33x35 cm	ST105046		THE ARTICLE NUMBER	
10	Seat ErgoMedic 36x39 cm	ST104546		ст101546	
10	Seat ErgoMedic 36x43 cm	ST104046		Article No: ST101546 "ErgoMedic seat, plush grey 44	_{4x} 48 cm"
10	Seat ErgoMedic 40x39 cm	ST103546		"Ergomedic seas, F	
10	Seat ErgoMedic 40x43 cm	ST103046		Digits 1-2 = Type of upholstered	d article
10	Seat ErgoMedic 40x48 cm	ST102546		Digite 3-4 = Seal System	
10	Seat ErgoMedic 44x43 cm	ST102046		Digits $5-6 = Size$	
10	Seat ErgoMedic 44x48 cm	ST101546		Digit 7 = Material	
10	Seat ErgoMedic 48x48 cm	ST101046		Digit 8 = Colour	



ErgoMedic® PLUS UPHOLSTERY FOR REAL 9000, 9000 PLUS, 6100, 6100 PLUS

BACKREST

The ErgoMedic Plus-back is available in several variants. The higher backrest provides more shape around the pelvis and thoracic spine, whereas the lower backrest has the same shape, but supports the pelvis and the lumbar spine.

The lower part of the high backrest is shaped so that you can sit with pelvic support even when tilted backwards. At the top in the centre, between the shoulder blades, a soft extra support has been integrated. This to enable the user to sit upright and stable against the backrest, whilst at the same time leaving enough room for free movement of the shoulders.

The high backrest is also available with extra side wedges. The low backrest has a similar shape to the high one, but is designed for supporting the pelvis and lumbar spine.

SEAT

The biggest requirement of the seat is to be stable and comfortable. ErgoMedic Plus-seats have the right form and softness to ensure optimal stability and comfort, higher under the thigh than the buttocks. Its form distributes pressure evenly and effectively, and prevents the user from sliding forward when the seat is angled.

When the user sits at the back of the seat, so that the pelvis is firmly placed against the lower parts of the back support, their legs are also stabilized. The higher rounded shape on either side of the thighs means that both the pelvis and the legs are stabilized.

ARMREST

The user should be able to switch between active and relaxed positions. The ErgoMedic Plus-armrests are shaped so the arms are supported from the sides, which means that the upper body is more relaxed. They are also raised beneath the wrists to provide extra relief. At the front, the armrest has a rounded convex shape, which makes it comfortable to rest the hand on, while providing a good grip when you want to get in or out of the chair.

NECKREST

The neckrest provides support primarily against the rear, lower part of the head. The lower part of the neck rest is built up with soft padding on each side, providing support for the lower skull and neck.

The upper part is flatter, which leaves room for the user to turn their head. The shape and properties of the neck rest means that the head is well stabilized, even when the user is sitting upright.



*8. ErgoMedic Plus Comfort-backrest is especially assembled for maximum comfort. It has an adjustable lumbar support (pump) which provides extra support in this region.

Bad	krest ErgoMedic PLUS, width x depth	Article No.	Sea	t ErgoMedic PLUS, width x depth	Article No.
1	Backrest ErgoMedic Plus 35x20 cm	BR208046	9	Seat ErgoMedic Plus 40x43 cm	ST203046
1	Backrest ErgoMedic Plus 40x20 cm	BR207546	9	Seat ErgoMedic Plus 40x48 cm	ST202546
2	Backrest ErgoMedic Plus 30x34 cm	BR203546	9	Seat ErgoMedic Plus 44x43 cm	ST202046
3	Backrest ErgoMedic Plus 35x43 cm	BR202046	9	Seat ErgoMedic Plus 44x48 cm	ST201546
4	Backrest ErgoMedic Plus 42x43 cm	BR201546	9	Seat ErgoMedic Plus 48x48 cm	ST201046
4	Backrest ErgoMedic Plus 46x43 cm	BR201046		<u> </u>	
5	Backrest EM Plus w wedge 30x34 cm	BR253546	Sea	t Coxit ErgoMedic PLUS, width x depth	Article No.
6	Backrest EM Plus w wedge 35x43 cm	BR252046	10	Seat Coxit Int EM Plus 40x39 cm	ST233546
6	Backrest EM Plus w wedge 42x43 cm	BR251546	10	Seat Coxit Int EM Plus 40x43 cm	ST233046
6	Backrest EM Plus w wedge 46x43 cm	BR251046	10	Seat Coxit Int EM Plus 40x48 cm	ST232546
7	Backrest EM Plus reinf wedge 35x43 cm	BR352046	10	Seat Coxit Int EM Plus 44x43 cm	ST232046
7	Backrest EM Plus reinf wedge 42x43 cm	BR351546	<u>10</u>	Seat Coxit Int EM Plus 44x48 cm	ST231546
7	Backrest EM Plus reinf wedge 46x43 cm	BR351046	10	Seat Coxit Int EM Plus 48x48 cm	ST231046
8	Backrest EM Plus Comfort 35x43 cm	BR212046			
8	Backrest EM Plus Comfort 42x43 cm	BR211546	Nec	krest, width x height	Article No.
<u> </u>	Back cot Ein i ac Comon 12x 10 cm		1100		
8	Backrest EM Plus Comfort 46x43 cm	BR211046	11	Neckrest cushion EM Plus 30x23 cm	NS2046
	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43				NS2046 NS4046
8	Backrest EM Plus Comfort 46x43 cm	BR211046	<u>11</u>	Neckrest cushion EM Plus 30x23 cm	
8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43	BR211046 BR262046	<u>11</u> <u>12</u>	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest	
8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43	BR211046 BR262046 BR261546 BR261046	<u>11</u> <u>12</u>	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm	NS4046
8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43	BR211046 BR262046 BR261546 BR261046 BR362046	11 12 Arm	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm	NS4046 Article No.
8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546	11 12 Arm 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm	NS4046 Article No. AR205138
8 8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546	11 12 Arm 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm	NS4046 Article No. AR205138 AR205238
8 8 8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43 Back EM Plus Comfort reinf wedge 46x43	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm	NS4046 Article No. AR205138 AR205238 AR202138
8 8 8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546 BR361046	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm	NS4046 Article No. AR205138 AR205238 AR202138
8 8 8 8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43 at ErgoMedic PLUS, width x depth Seat ErgoMedic Plus 29x32 cm Seat ErgoMedic Plus 33x35 cm	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546 BR361046 Article No. ST205546 ST205046	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm	NS4046 Article No. AR205138 AR205238 AR202138
8 8 8 8 8 8 8 8	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43 at ErgoMedic PLUS, width x depth Seat ErgoMedic Plus 29x32 cm Seat ErgoMedic Plus 33x35 cm Seat ErgoMedic Plus 36x39 cm	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546 BR361046 Article No. ST205546 ST205046 ST204546	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm Armrest ErgoMedic Plus Left 30 cm	NS4046 Article No. AR205138 AR205238 AR202138 AR202238
8 8 8 8 8 8 8 9	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43 at ErgoMedic PLUS, width x depth Seat ErgoMedic Plus 29x32 cm Seat ErgoMedic Plus 33x35 cm Seat ErgoMedic Plus 36x39 cm Seat ErgoMedic Plus 36x43 cm	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546 BR361046 Article No. ST205546 ST205046	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm Armrest ErgoMedic Plus Left 30 cm underlined article appears in the pi	NS4046 Article No. AR205138 AR205238 AR202138 AR202238
8 8 8 8 8 8 9 9 9	Backrest EM Plus Comfort 46x43 cm Back EM Plus Comfort w wedge 35x43 Back EM Plus Comfort w wedge 42x43 Back EM Plus Comfort w wedge 46x43 Back EM Plus Comfort reinf wedge 35x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 42x43 Back EM Plus Comfort reinf wedge 46x43 at ErgoMedic PLUS, width x depth Seat ErgoMedic Plus 29x32 cm Seat ErgoMedic Plus 33x35 cm Seat ErgoMedic Plus 36x39 cm	BR211046 BR262046 BR261546 BR261046 BR362046 BR361546 BR361046 Article No. ST205546 ST205046 ST204546	11 12 Arm 13 13 13	Neckrest cushion EM Plus 30x23 cm Neckrest cushion EM Plus 30x8 cm nrest Armrest ErgoMedic Plus Right 42 cm Armrest ErgoMedic Plus Left 42 cm Armrest ErgoMedic Plus Right 30 cm Armrest ErgoMedic Plus Left 30 cm	NS4046 Article No. AR205138 AR205238 AR202138 AR202238



OTHER UPHOLSTERED PARTS

TRUNK SUPPORT

The trunk support is used to produce more upright, active sitting. Mercado has three types: curved and adjustable, straight, and curved trunk support.

NECK REST

The neckrest provides support for the head in order to achieve the best sitting position and to provide relief for the neck and spine even for users with poor body control. It can easily be adjusted to achieve the best setting for the individual user. Available in flat, curved or ErgoMedic Plus-versions.

COVER

The covers protect the upholstery from dirt and dust, as well as extending the upholstery's lifetime. Covers are available in different colours for a variety of combinations. Incontinency covers are a quick and easy solution to protect seats against moisture.

ARMREST

The Hemiplegi-armrest is suitable for partially paralysed, or users with restricted control. The armrest panel has been hollowed out, which makes it easier to keep the paralysed arm on the rest. The armrest panel also rotates, which allows for the best possible position. The mobile armrest panel is upholstered with extra foam for optimised comfort.

SIDE SUPPORT

The side supports help the user maintain stability and position. These can also be used as thigh supports, so that the users feet are better supported on the footrests or the floor, and prevents their legs from sliding apart. It can be adjusted for height and width, or turned backwards and forwards. Available in three sizes: S, M and L.

SLOW RECOVERY SEAT AND OTHERS

Slow Recovery-seats are manufactured with two materials in the cushioning, which provides optimum support and comfort to the user.

The upper layer comprises of a memory foam that moulds itself, and provides good pressure distribution and increased sitting comfort, while the lower layer raises the seat bone nodules from the wood plate. Even if the user does not sit in the chair for a sufficiently long period of time, the seat cushion adjusts itself so that a cavity is formed where the person sits. This material is available for all upholstered parts.

The wooden seat disc should be used when casting or retrofitting of its shell. The seat with anti-slip is recommended when using individually adjusted seats. Seat with hygiene fabric should be used by users with incontinence problems. This seat is to be completed with a cover in plush.



Trunk support, width x height		Article No.	Neckrest, width x height Article No.					
	1	Trunk support cushion 1	5x13 cm	TS1046	3	Neckrest cushion	Straight 29x16 cm	NS1046
	2	Trunk support curve- an	d adjustable	TS2038	4	Neckrest cushion	Curved 31x18 cm	NS3046
Armrest				Arti	cle No. Article N	lo. for angle		
	6	Armrest ErgoMedic 25	cm	AR101038	240	387		
	6	3		AR102038	240386			
	6	9		AR103038	240385			
	6	<u> </u>		AR104038	240388			
	7 Armrest Mobile 25 cm			AR301038	245387			
	7 Armrest Mobile 30 cm		AR302038	245386				
	7	7 Armrest Mobile 35 cm		AR303038	245385			
	7	7 Armrest Mobile 40 cm		AR304038	245388			
	8	8 Armrest Hemiplegic 45x12 cm		AR406038	244387			
	9	Wooden seat disc		ST10xx90	Cov	ers		
	10	Wooden seat disc with	anti-slip	ST10xx61	5	Exist in plush, p	atterned fabric and	incontinence
	Side	e support, width x heigh	nt	Article No.	material. Please see www.mercado-medic.co.uk			
11 Side support cushion XS, 15x8 cm		SS1046	or ontact Mercado for more informat		ation.			
	12 Side support cushion S, 28x8 cm		SS2046					
13 Side support cushion M, 24x14 cm 14 Side support cushion L, 30x16 cm		SS3046	16 Seat ErgoMedic with hygiene fabric. To be					
		SS4046 completed with cover in plus		cover in plush.				
	Sea	t Slow Recovery	Slow Recovery	Slow Recovery	Slo	w Recovery	Slow Recovery	
		e in width x depth	ErgoMedic	ErgoMedic Coxit		oMedic PLUS	ErgoMedic PLUS	Coxit
	15	33x35 cm	ST115046	-	ST2	15046	-	
	15	36x39 cm	ST114546	-	ST2	14546	-	
	15	36x43 cm	ST114046	-	ST2	14046	-	
	15	40x39 cm	ST113546	ST143546	ST2	13546	ST243546	
	15	40x43 cm	ST113046	ST143046	ST2	13046	ST243046	
	15	40x48 cm	ST112546	ST142546	ST2	12546	ST242546	
	15	44x43 cm	ST112046	ST142046	ST2	12046	ST242046	
	15	44x48 cm	ST111546	ST141546	ST2	11546	ST241546	
	15	48x48 cm	ST111046	ST141046	ST2	11046	ST241046	



UPHOLSTERY

FOR REAL 2000, 9500/9600 EL, 9200 TWIN/EL

To rehabilitate the body after a serious accident is usually a long process. With proper rehabilitation, this process can be shortened. A prerequisite for short rehabiltering is the availability of good ergonomic aids. An ergonomically correct seating system helps reduce the patient's pain following, for example, whiplash accidents or to find relief when standing/sitting work. This seating system also works great when the chairs are used as a technical means (e.g. feeding situations).

SEAT SADDLE CLASSIC 36X36 AND 41X41

Plan saddle seat "all-rounder", fits most.

SEAT SADDLE MEDIC 40X36

Rounded saddle seat that offers good positioning.

SEAT SADDLE ERGOMEDIC 40X36

Partially buried in the back, thus some relief for the tailbone.

SEAT SADDLE ERGOMEDIC 40X36, LOWERED NOSE, LOWERED FOR TAILBONE

Provides relief of tailbone and spine at the appropriate use of chest support.

SEAT SADDLE ERGOMEDIC LOWERED NOSE, PART SAWN REAR

Allows for maximum relief of tailbone, vertebral column and the "seat bones" at the appropriate use of chest support.

ROUND, CONICAL SEAT

The round, conical shape makes it possible to vary its position on the placement of bone and degree of pelvic

tilt. The shape also means reduced pressure under the thighs at the appropriate use of chest support.

STOMACH SUPPORT WITH DOUBLE TIP

Stabilizes a larger area which is often requested by impairments neck and thoracic spine.

The slightly thicker padding enable for thin people to find a suitable position.

STOMACH SUPPORT MEDIC TIP UP

Stabilization of the thoracic or chest area/low back when only small chest support is required.


STOMACH SUPPORT MEDIC TIP DOWN

Allows for stabilization of the lumbar/pelvic and leaving great freedom of movement for the legs.

The straight edge at the top is comfortable to rest arms against.

REAL 9200 TWIN/EL - EXTRA PADDED SEATS

Being overweight is often problematic in several ways. Extra padded seats provide relief.

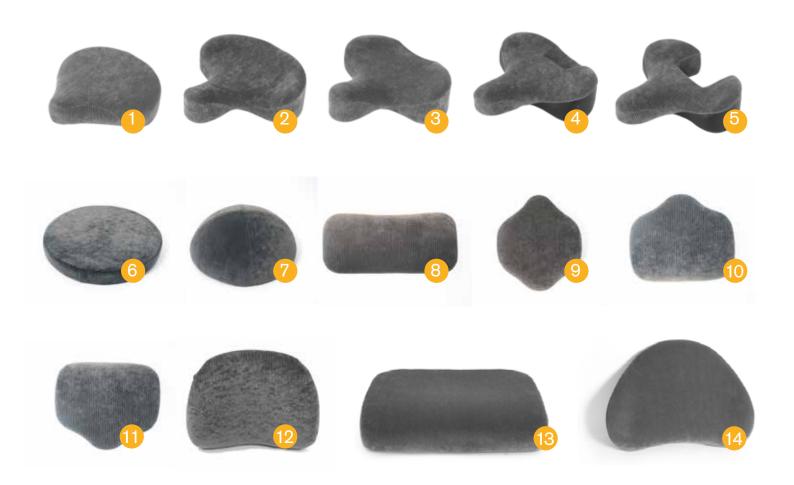


REAL 2000, 9500/9600 EL			RE/	REAL 9200 TWIN/EL				
Seat, width x depth		t, width x depth	Article no.	Seat, width x depth		Article no.		
	1	Seat Saddle Classic 36x36	ST324746	13	Seat Twin 72x50	ST406046		
	1	Seat Saddle Classic 41x41	ST322346	13	Seat Twin 62x50	ST406346		
	2	Seat Saddle Medic 40x36	ST333646	13	Seat Twin 52x50	ST406646		
	3	Seat Saddle ErgoMedic 40x36	ST343646	13	Seat Twin 72x45	ST406146		
	4	Seat Saddle Sawn f. Tailbone 40x36	ST363646	13	Seat Twin 62x45	ST406446		
	5	Seat Saddle part sawn Rear 40x36	ST373646	13	Sits Twin 52x45	ST406746		
	6	Seat Round Diameter 35	ST305146	13	Seat Twin 72x40	ST406246		
	6	Seat Round Diameter 40	ST303846	13	Seat Twin 62x40	ST406546		
	7	Seat Round Diameter Conical 35	ST315146	13	Seat Twin 52x40	ST406846		
Backrest and Stomach support, width x height			Bac	krest, width x height				
	8	Backrest Classic 33x16	BR109046	14	Backrest Twin 66x43	BR500646		
	9	Stomach support double tip 27x27	BR416746	14	Backrest Twin 65x42	BR500846		
	10	Stomach support Medic 27x23	BR406946	14	Backrest Twin 46x43	BR501046		
	11	Stomach support Medic 27x23	BR406946					
	12	Backrest ErgoMedic 33x26	BR106046					


FITTED ARTICLES

Mercado produces all upholstered furnishings inhouse. We can as well manufacture these in any special size, material, clothing, etc. (please see page 22).


COVERING

PLUSH

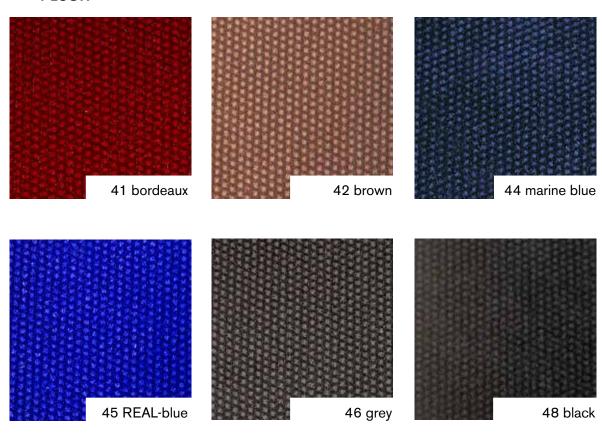

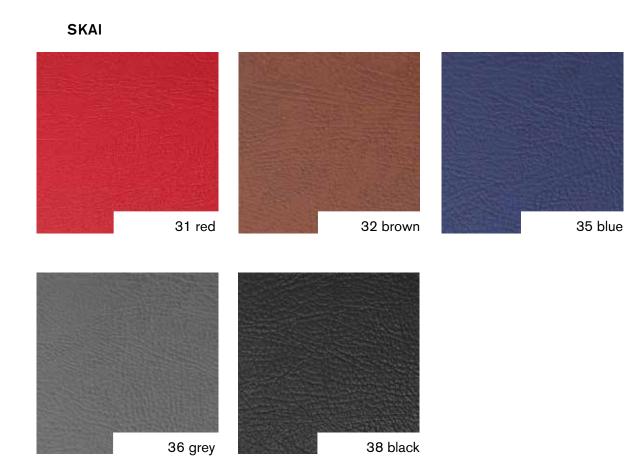

PATTERNED FABRIC ADULT

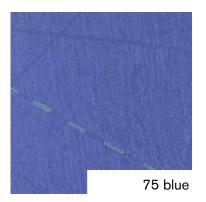

EXPLANATION OF THE STRUCTURE OF THE ARTICLE NUMBER

Article No: ST101546


"ErgoMedic seat, plush grey 44x48 cm"

Digits 1-2 = Type of upholstered article Digits 3-4 = Seat system
Digits 5-6 = Size

Digit 7 = Material

Digit 8 = Colour

PATTERNED FABRIC CHILD

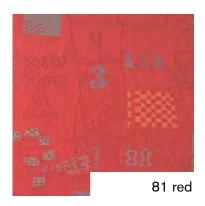






ALTERNATIVE COVERING





* Coverings are available in these colour codes; 41, 42, 44, 45, 46, 48, 75, 81, 85, 89, 66, 73, 74. Please go to www.mercado.se or contact us for more information.

COVERING*

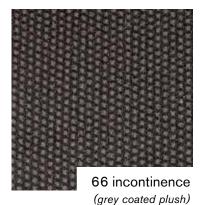





FITTED ARTICLES

Mercado manufactures all upholstered furnishings inhouse. We can also manufacture these in any special size, material, clothing, and more. Download forms for adaptation from the website, www.mercado.se, under "Documents".

